

Simulator Name: MetiMan

Simulator Type: MetiMan, Fully wireless

Vital Signs:	<ul style="list-style-type: none">• Heart rate• Respiratory rate• Blood Pressure• Pulses• Temperature cannot be taken on simulator but can be displayed on monitor if relevant to the scenario
Physical Assessment:	<ul style="list-style-type: none">• Voice• Pulses: are touch activated<ul style="list-style-type: none">▪ Bilateral carotid▪ Bilateral brachial▪ Bilateral radial▪ Bilateral femoral▪ Bilateral popliteal▪ Bilateral dorsalis pedal▪ Posterior tibiali• Heart Sounds<ul style="list-style-type: none">▪ Aortic area▪ Pulmonary area▪ Tricuspid area▪ Mitral area• Breath sounds on upper and lower pulmonary areas<ul style="list-style-type: none">▪ Normal vesicular sounds▪ Crackles▪ Wheezes▪ Diminished▪ Gurgling▪ Pleural rub▪ Rhonchi• Blood pressure• Reactive eyes with automatic and manual, consensual and independent papillary response and control, blinking eyes, variable pupil size• Bilateral chest movement, Intercostal palpable ribs, flail chest, Pneumothorax decompression, Independent chest excursion• Bowel sounds on four quadrants• Realistic airway, break away teeth, trismus, trachea with realistic anatomical landmarks• Realistic skin, cyanosis, capillary refill (Thumbs)• Convulsions
Skills:	<ul style="list-style-type: none">• Cardiac and pulmonary auscultation• Blood pressure assessment• Pulse palpation• Bilateral IV port for IV access• Chest compressions• Central Venous catheter (subclavian) unilateral• Nasogastric tube placement• Tracheostomy tube placement• 5 Lead ECG, defibrillation, pacing• Intubation• Dressings (Must be adhered to simulator with special "Mepitac" tape)
Enhance Your Experience:	<ul style="list-style-type: none">• Realistic secretion: Eyes, nose, mouth• Two simultaneous bleeding sites• Urinary output for indwelling urinary catheter• Interchangeable female and male genitalia <p style="text-align: center;">Note: the setting up of those features requires at least one hour preparation</p>
Important Information:	<ul style="list-style-type: none">• Keep excessive care when moving to protect internal circuit and mechanics• Keep ink, marking materials, printed materials away from the skin

